



# Handpiece Repair and Maintenance Guide

- 1 Disconnect your handpiece from the dental unit
- 2 Remove the bur whilst wearing gloves
- 3 Wash the handpiece with cold water
- 4 Shake the handpiece maintenance spray can vigorously
- 5 Holding the can upright, insert the nozzle into the base of the handpiece. Ensure the correct related nozzle is used
- 6 Spray for 1-2 seconds into a tissue
- 7 Purge (run) the handpiece for 20 – 30 seconds or spray until the excess oil being blown into the tissue is clean
- 8 Wipe off excess oil with a dry tissue
- 9 Place handpiece into the autoclave - Do not use pouches unless you are using a vacuum autoclave
- 10 Select the correct cycle -Do not use a drying cycle in a non vacuum autoclave
- 11 When the cycle is complete take the handpiece out immediately

