


handpiece hotline

The World's Best Handpiece Repair Service

Handpiece Repair & Maintenance Guide

1. **Disconnect your handpiece from the dental unit**
2. **Remove the bur whilst wearing gloves**
3. **Wash the handpiece under cold water**
4. **Shake the handpiece maintenance spray can vigorously**
5. **Holding the can upright, insert the nozzle into the base of the handpiece. Ensure the correct related nozzle is used**
6. **Spray for 1-2 seconds into a tissue**
7. **Purge (run) the handpiece for 20 – 30 seconds or spray until the excess oil being blown into the tissue is clean**
8. **Wipe off excess oil with a dry tissue**
9. **Place handpiece into the autoclave -
Do not use pouches unless you are using a vacuum autoclave**
10. **Select the correct cycle -
Do not use a drying cycle in a non vacuum autoclave**
11. **When the cycle is complete take the handpiece out immediately**

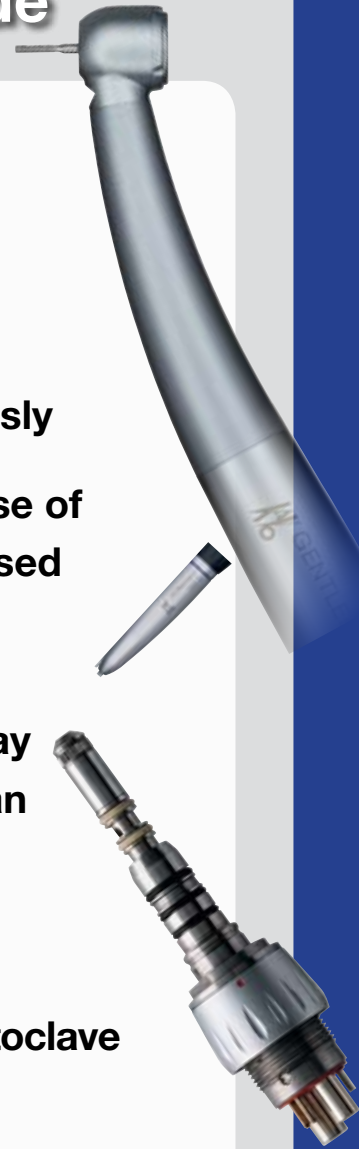

Call us for a freight FREE handpiece repair bag

P: 1800 30 80 30

F: 02 9998 5050

E: service@handpiecehotline.com.au

W: www.handpiecehotline.com.au